

MK-1028 R1 InMotion System Cleaning Procedure Summary

InMotionRobots™

JOIN THE NEURORECOVERY REVOLUTION

The Bionik Team takes your patient care needs seriously, and in this time of uncertainty, we want to assure maximum safety for you and your patients. With that in mind, we would like to remind all our partners about the proper cleaning procedures for the InMotion ARM and ARM/HAND systems.



MK-1028 R1 InMotion System Cleaning Procedure Summary

BIONIK

InMotionRobots[™]

This is a summary of recommended cleaning instructions for all patient and/or clinician contact areas of the InMotion Systems. This summary is in reference to the detailed WI-1024 R1 System Cleaning Procedure.

Please utilize the following materials for cleaning your InMotion System:

- PDI Super Sani-Cloth Disposable Wipes (P/N Q55172 or equivalent)
- PDI Sani-Cloth Bleach Germicidal Disposable Wipes (P/N P54072 or equivalent)
- Medline Sterile Cotton Tip Application, 6" (P/N MDS202095 or equivalent)
- Isopropyl Alcohol, 70% (P/N MCHEM145 or equivalent)
- Low-Lint Wipes (TechniCloth® TX609 or equivalent)

Procedure:

- <u>Recommendation</u>: Cleaning of the InMotion Systems shall occur immediately following contact with the device with any individual (patients, clinicians, etc.). PDI Sani-Cloth Bleach Germicidal Disposable Wipes recommended for appropriate patient conditions to adhere to hospital infection prevention protocol. Follow manufacturer directions for all cleaning materials and consult the available InMotion User Manual for additional information on accessory locations/ diagrams.
- 2. Wipe down external surfaces of ARM accessories such as Forearm Support, Arm Handle, Wrist Support & Pronation/Supination Adjustment Latch using PDI Super Sani-Cloth Disposable Wipes.
- 3. Wipe ARM/HAND accessories such as Forearm Support, exterior and interior surfaces of Finger and Thumb Grips, Hand Rotation Buttons and Pronation/Supination Adjustment Latch.
 - Interior holes of Finger and Thumb Grips shall be cleaned by inserting Medline Sterile Cotton Tip Application (6") soaked in Isopropyl Alcohol (70%). Wipe all excess Isopropyl Alcohol (70%) with Low-Lint Wipes.
- 4. Wipe Worksurface Areas, Patient Chair, Monitor, Keyboard, Printer including Power Cord using PDI Sani-Cloth Bleach Germicidal Disposable Wipes.